


Performing the A1cNow Test

The 4-test Box

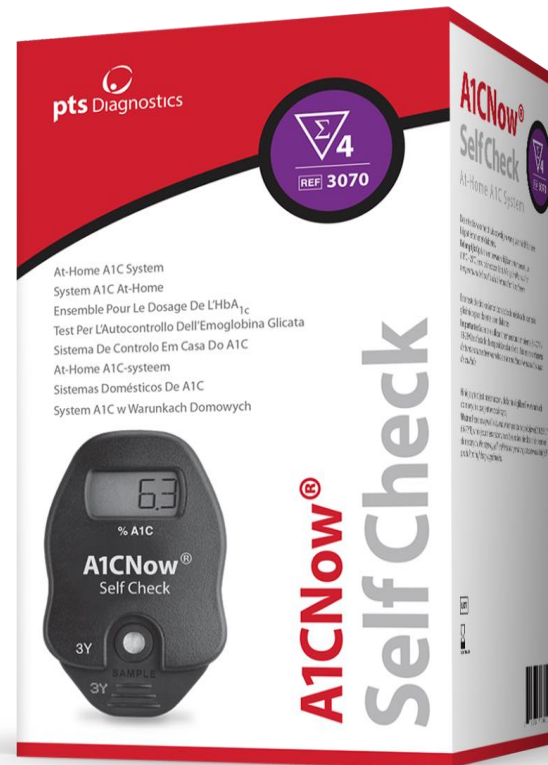

Inside The Box



Meter and Foil Pouch

Foil Pouch contains

- A lancet to use for fingerstick (not shown)
- The clear plastic blood collection tube
- The Black Dilution Tube
- The Test Strip

Collecting Blood and Diluting It



1. Prick Finger and Collect blood to fill small tube

2. Place collector into the **BLACK** end of Dilution Tube. Mix by rotating 5-7 times

3. Insert test strip into meter and apply sample as shown

Place Test Strip In Meter



Read Test Result in 5 Minutes

