

Performing the A1c

Home Test



The 4-testBox

Meter and Foil Pouch

Foil Pouch contains

A lancet to use for fingerstick (not shown)

The clear plastic blood collection tube

The Black Dilution Tube

The Test Strip

Inside The Box

4 finger lancets

1 Meter



4 Test Strips



4 Dilution Tube



4 Blood collector


Collecting Blood and Diluting It




- Prick Finger and Collect blood to fill small tube
- Place collector into the **BLACK** end of Dilution Tube. Mix by rotating 5-7 times
- Insert test strip into meter and apply sample as shown

Place Test Strip In Meter

Meter

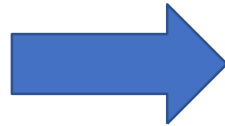

Test Strip

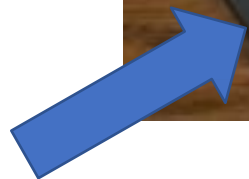

Read Test Result in 5 Minutes



Warning – machine shuts off after 5 minutes and result cannot be recovered, so stay until you get the test result